



Options for EGFR, KRAS or BRAF Sample Preparation

OPTION 1

Tissue block sent to Healthscope Pathology for processing.

Send Formalin-fixed paraffin embedded tumour sample to:

*Molecular Pathology
ATTN: Dr Keith Byron
Healthscope Pathology
1868 Dandenong Rd
Clayton VIC 3168*

Once results have been received by the referring clinician, the tumour sample will be returned to the originating laboratory.

OPTION 2

Processing performed by laboratory where the tissue block is held.

Using a new microtome blade, prepare seven unstained slides at a depth of 7 microns each and one H & E stained section.

Label all slides with the pathology number.

Place the labeled sections on slides into slide mailers and send to:

*Molecular Pathology
ATTN: Dr Keith Byron
Healthscope Pathology
1868 Dandenong Rd
Clayton VIC 3168*

A delay in sample processing may delay patient treatment.

Contact one of our product specialists on **0402 563 020** or **0424 750 403**

If you have any questions or require further information, please contact our Customer Service Centre.

1300 453 688

Providing services since 1936, Healthscope Pathology is part of Healthscope Ltd., a publicly listed company that employs over 18,000 people throughout Australia, New Zealand, Singapore and Malaysia.

Healthscope has purpose built, state-of-the-art service and molecular research laboratories which ensure the highest standard of testing.

focused on you

healthscopepathology.com.au

HAP-1175-W-001 (08/10)
Clinical Laboratories Pty Ltd ABN 62 006 823 089